TRIOTM
CLEAR

Progressive Clear
Aligner System

Clear Aligner
Everyday Guide

This brochure serves as a quick reference guide.
For more details, please refer to the "Instruction For Use"
included with the product.

Rx Only - For professional dental use

Your confident
smile starts here
with TrioClear™



LANGUAGES

English	4
Simplified Chinese	8
Traditional Chinese	12
Danish	16
Dutch	20
Finnish	24
French	28
German	32
Korean	36
Norwegian	40
Spanish	44
Swedish	48
Thai	52

Q How to insert the aligners?

1. **Wash your hands** thoroughly with soap and water before handling your aligners.
2. Wear the aligners with a specific sequence step of **Soft (S)** and **Hard (H)**.
3. **Clean the aligners** with a soft toothbrush and lukewarm water or mild soap prior to using them.
4. Insert either the upper or lower aligner first. **(a)** When inserting each aligner, gently push the aligner over your incisors. **(b)** Then apply pressure using your fingertips, to the top of the left and right molars until the aligner snaps into place.
5. **Use the TrioKey for at least 5 minutes** every time you put on the aligners.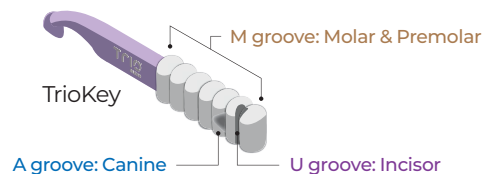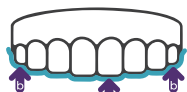

6. Position the TrioKey between your teeth firmly on all surfaces of your teeth and bite down.

Incisor-U groove

Put your top teeth into the U groove on the TrioKey and bite your lower teeth against it a few times. Then fit your bottom teeth into the U groove and bite your top teeth on the TrioKey a few times.

Canine-A groove

Put your top teeth into the A groove on the TrioKey and bite your lower teeth against it a few times. Then fit your bottom teeth into the A groove and bite your top teeth on the TrioKey a few times. Repeat on the other side of your mouth.

Molar and Premolar-M groove



Put the M groove on the TrioKey between your back teeth on one side of your mouth and bite up and down a few times. Repeat on the other side of your mouth.

7. The recommended wearing time varies from case to case, please follow your doctor's instructions.

Q How to remove the aligners?

1. Wash your hands thoroughly with soap and water before handling your aligners.
2. Using your fingers or the TrioKey, **(a)** start by disengaging the aligners at the molar regions on both sides. **(b)** Then slowly work your way to the front with balanced forces to fully remove.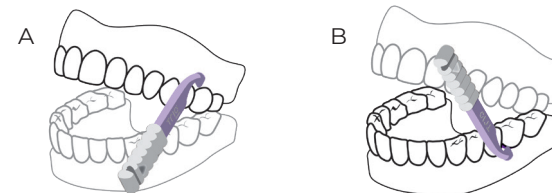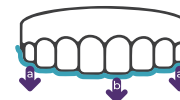

A. To remove a aligner from your upper teeth:

Position the TrioKey so that the end of the TrioKey is parallel to the upper aligner. Latch the TrioKey from the furthest molar, gently pull down and forward.

B. To remove a aligner from your lower teeth:

Position the TrioKey so that the end of the TrioKey is parallel to the lower aligner. Latch the TrioKey from the furthest molar, gently push up and forward.

Aggressive or improper use of the TrioKey can result in breakage of the aligner, if you are having difficulties in one area, move the TrioKey to a new position and try again. Make sure that the TrioKey is latched onto the aligners only to avoid damage to gums, teeth or dental prosthesis.

3. Finally, rinse the aligners every time you remove the aligners and keep the aligners cool and dry into the given aligner case.

Q How to preserve your lifelong smile after the treatment?

Retainers are worn to prevent teeth from shifting back to their original positions after the treatment. Teeth also tend to shift over time as we grow older, hence wearing retainers can preserve your lifelong smile.

Ecodont™ Retainer is a doctor-prescribed, custom manufactured, clear plastic removable orthodontic appliance that you will wear after getting your teeth straightened, whether with the TrioClear™ treatment, traditional braces, or other conventional aligners.

For more information, please visit www.TrioClear.com/ecodont-retainers or consult your doctor.

Important Notes

1. Do not bite the aligners in place, use finger pressure to fit the aligners in every time. If you experience sharp pain or significant discomfort, please contact your doctor immediately.
2. If you suspect an adverse reaction, discontinue use and consult your doctor. Your doctor, not TrioClear™, is responsible for managing your treatment.
3. Aligners should be removed for eating and drinking, except for cool water.
4. To help prevent damage, avoid unnecessary removal.
5. Do not use excessive force to bend or twist an aligner to remove.
6. Always keep the most recently used aligners (or more pairs of aligners as directed by your doctor). If your current aligner is lost or broken, you should contact your doctor immediately. Your doctor may recommend that you temporarily go back to the previous pairs before getting a replacement.

Frequently asked questions

What is the Progressive Clear Aligner System?

TrioClear™ is a Progressive Clear Aligner System. From SOFT to HARD, the varying levels of thickness from 0.5mm to 0.7mm shifts the teeth comfortably and help each patient achieve their desired smiles.

Why do my teeth feel pressure during the treatment?

It is normal for your teeth to feel pressure at the beginning stage. This means that the aligners are starting to move your teeth. The feeling should disappear gradually in a couple of days. If this is not the case, please contact your doctor immediately.

Why do I lisp when I wear the aligners?

In the first few days you may speak with a slight lisp, which will go away after several days as your tongue adjusts to your aligners.

How can I clean the aligners?

You may gently brush the aligners with a soft toothbrush and lukewarm water or mild soap. Please remember not to use toothpaste or denture cleanser or other chemical solvents such as bleach or rubbing alcohol to clean the aligners.

How can I store the aligners?

You should avoid placing the aligners in places of high temperature and humidity, and keep the aligners out of reach of children and pets.

How can I dispose of the aligners?

Clear aligners and other such appliances naturally pick up a lot of oral bacteria, and sterilization is needed before disposal. In order to dispose of the appliances properly, you may check the local Waste Disposal Department since waste management varies in different countries.

For more information, please visit www.TrioClear.com/faqs-patients

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© 2024 TrioClear. All rights reserved.